

ABL2WDC Patch Installation Instructions

This patch converts existing ABL2WDC installations for use with later versions of Windows 95, Windows 98 and Windows NT Version 4.0. When the install program on this disk is run, it copies specific files to the default directory for ABL2WDC. An existing installation of ABL2WDC must be present on your computer prior to the installation of this patch.

Installation:

1. Click *Start*.
2. Click *Run*.
3. In the dialog box, input *A:\Install*.
 - a. Windows NT users: Click the checkbox *Run in Separate Memory Space*.
4. The installation program offers an opportunity to change the destination directory.
 - a. Accept the default directory or change the destination if your installation was customized to use another directory.
5. Click *Continue* or press *Enter*.
 - a. The program copies the ABLVIEW data conversion engine and associated files to the directory specified in the previous step.
 - b. The program informs you the ABL2WDC has been properly installed on your system.
6. Click *OK* or press *Enter* to terminate the installation program.

The installation of this patch is now completed. Test this installation by following the ABL2WDC instruction for making WDC submission files.

If you have any questions regarding this patch or its installation, please contact our Technical Support Department at **1-800-736-0600, ext. 6124**.