

Downloading the ABL2WDC Program

***Please have your IT staff download the program
and install it on your PC.***

*****Important*****

***Have four formatted high-density floppy disks available before
downloading the program.***

1. Go to the Radiometer site <http://www.radiometeramerica.com/>.
2. Click on Download under WDC online
3. Click on *Download ABL2WDC_1*
A File Download Window will appear
 - a. Click on Save
 - b. Name the file ABL2WDC_1.exe
 - c. Select a destination to save the file
 - d. Click on Save
4. Save each of the remaining files to the same location using ABL2WDC_2.exe, ABL2WDC_3.exe, ABL2WDC_4.exe and PATCH.exe for the names.
5. Locate the files that were saved
6. Insert a blank disk in the floppy drive
7. Click on the ABL2WDC_1.exe
A WinZip Self-Extractor Box will appear
The default *Unzip to folder* location is A:\.
Please change the folder location if extracting to a different location.
8. Click on *Unzip*
The data will be unzipped to the floppy drive unless the folder location was changed.
8. Click *OK*.
8. Close the WinZip Self-Extractor box.
9. Follow the same procedure for ABL2WDC_2, ABL2WDC_3 and ABL2WDC patch.